

# Instruction Manual

## Affinity Diva Spa Pro

AFDIVASPRO/B



## 1. Specifications

## 2. Instructions

How to Adjust the Couch Height

How to Adjust the Back and Leg Rests

How to Install/Remove the Side Arm Rests

How to Install/Remove the Face Cradle and Arm Rest Shelf

How to Move Your Massage Couch

## 3. Care and Maintenance


How to Care For Your Upholstery

General Maintenance

## 4. Warnings

# Specifications

**Affinity Diva Spa Pro**  
**Product Code: AFDIVASAPRO**

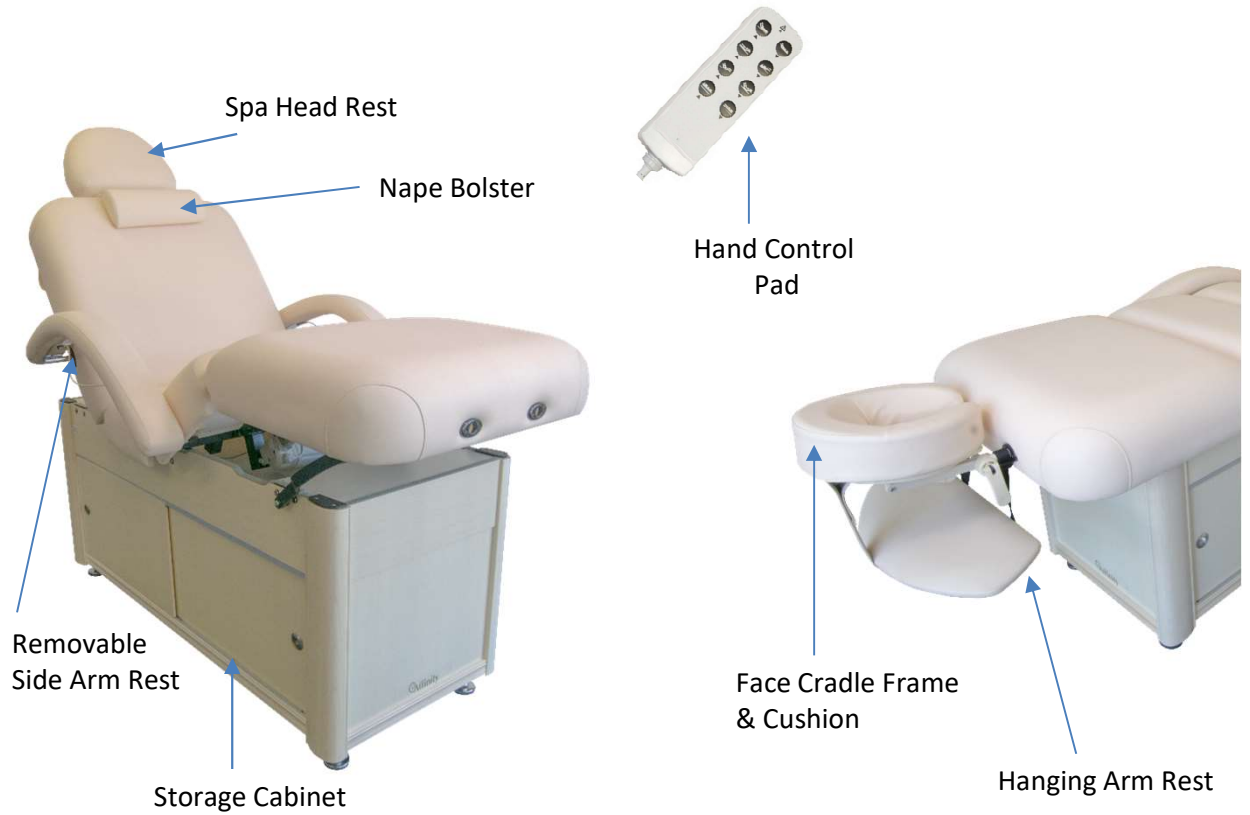

Height Adjustment:	71cm – 86.cm (28”– 34”)
Weight:	122kgs
Set up dimensions:	Width 71cm (28”)
Excl. face cradle & armrest:	Length 184cm (72.5”)
Incl. face cradle & armrest:	Length inc. headrest 211cm (83”)
Materials:	European beech wood frame, 4” Therafoam padding, PU vinyl.
Weight Capacity:	295kgs (46.5 stone)
Warranty:	2 years
Ports for Face Cradle/Paper Towel Holder:	Both ends.
Colours:	Biscuit 
Shipping W&Ds (approx):	W81cm x L188cm x H86.4cm. Weight 135.6kgs

# Instructions

## Adjust the Couch Height:

The couch height can be controlled via the hand held control pad.



Hand Held Control Pad



## Adjust the Back and Leg Rests:



Controls Back Rest

Controls Leg Rest


Controls Back and Leg Rest



## Install the Side Arm Rests:



Arm Rest Prongs

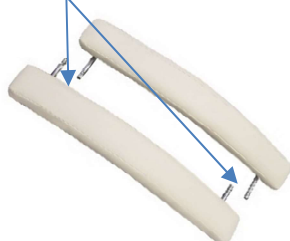

On the underside of the couch, beneath the side holes, you will find thumbscrews. Before trying to insert the arm rests please ensure the screws have been loosened.

Slot the arm rest prongs into the holes in the side of the couch and tighten the thumbscrews.

To remove the Side Arm Rests, loosen the thumbscrews and gently slide the arms out.

The Side Arm Rests do not need to be removed when changing the angle of the back and leg rests.



Do not sit on the Side Arm Rests.  
Do not place excessive weight on the Side Arm Rests.

# Instructions

## How to Use the Face Cradle and Hanging Arm Rest:



Face cradle cushion



Face Cradle frame



Hanging arm rest



Lay the hanging arm rest on the couch.



Lay the face cradle over the top and slide the black loops over the cradle prongs.



Slide the face cradle into the port holes on the couch.



Lock the face cradle into position using the lever.



Wrap the arm rest velcro strap around the face cradle frame.



Place the face cradle cushion onto the face cradle frame.



To change the angle of the face cradle frame, simply release the locking lever, move the cradle into desired position and lock the lever.

## How to Move Your Massage Couch:



Because the Diva Spa Pro weighs 122kg extreme caution should be taken when trying to move the couch.

A 4-man lift is recommended.

## How to Care for Your Upholstery:

In an effort to safeguard your product's quality and maintain the life of your couch, we advise the following:

- Avoid storing your table in extreme cold temperatures as the cold will cause cracking.
- Avoid storing your table in extreme hot temperatures. Heat will cause the upholstery to stretch and pucker. Keep your table away from heaters, electrical appliances and out of direct sunlight.
- Always protect your couch with accessories such as covers and sheets.
- Clean your upholstery daily, taking care to clean inside the breathe hole as even the oils naturally found on our skin will cause upholstery to break down. Regular cleaning will help to protect it against oils, stains and dirt, which, if left, will cause cracking and permanent stains.
- We suggest a mild mixture of anti bacterial detergent (avoiding those containing citric acid) and water in a spray bottle. Spray the table with the solution and wipe down the table with a clean, damp towel.



The use of alcohol will dry out the vinyl and cause it to crack.

Prolonged exposure to extreme temperatures and direct sunlight will cause damage to vinyl.

## General Maintenance:

Regularly check wires and cables for signs of wear or splitting.

In the event of noisy operation, please contact your retailer.

# Warnings



Risk of injury may be caused by improper set up, operation or maintenance of the equipment.



Please read this instruction manual BEFORE operating this couch.



Do check the padding and upholstery when unpacking the couch. Do not use if there is any sign of damage.



Lifting capacity (incl. client & accessories): 95kg/46.5 stones



For your safety, please do not try to disassemble the Hand Control, Control Box or Actuator.



Keep children and pets away from the electric couch. Do not go under the couch in case of any personal injuries caused by the collapse of the couch.



Do not put any items underneath the back-rest or leg-rest.



Do not put excessive weight or pressure on the face cradle or spa head rest.



Should the hand controller come into contact with water, cut the power to the couch immediately.



Do not expose the couch to extreme temperatures or humidity.



Do not lay the couch on its side without taking protective measures.

## Safety:

We recommend checking all connections and moving parts on a regular basis.

Contact your dealer in case of safety-relevant signs of wear.

Unauthorised modification or alteration of this couch is prohibited and will void the warranty.



Thank you for choosing Affinity.

Every care has been taken to ensure that this product reaches you in excellent working condition.

Should you have any questions or concerns please refer back to your original supplier who will be happy to assist.