



Bio-Therapeutic

bt-analyze®

INSTRUCTION MANUAL



Congratulations on your purchase of the bt-analyze! Your equipment has been built with the highest quality components for lasting precision and operating durability.

READ ALL INSTRUCTIONS BEFORE USE

Bio-Therapeutic products are not to be used to diagnose or treat any medical conditions.

HOW TO USE

Using the bt-analyze®, measure the moisture on up to seven locations on the face, forehead, crows feet, below the cheek bone, chin, and neck.

ZONE SPECIFIC ANALYSIS

Remove cover by pulling out and up from the bottom of the cap.

Turn on the bt-analyze by holding the power button for 2 – 3 seconds. The unit will calibrate itself. When the display screen is done flashing and the unit has beeped twice, the bt-analyze is ready for moisture readings.

Place the bt-analyze on the surface of the skin; a beep will signify the reading is complete. Press the memory key to save the reading on the top of the display screen. Readings will save in the order you perform them.

PLEASE NOTE: A soiled membrane slows calibration and reading times. Wipe the membrane without pressing down with a clean, dry tissue between each reading and with an alcohol wipe after each use. Always remember to replace the cap.

Note the moisture variances on different areas of the face. Educate clients of your findings as well as what bT-Ceuticals® products you recommend, along with home care instructions. At this time, also educate clients on recommended professional services to meet their skin care goals.

The bt-analyze membrane will make a slight click when depressed during moisture readings; this is normal. A reading of “UUU” means an error has occurred.

An error reading may occur during idle times when the unit is not in use. Simply press the memory button and previous memory will resume.

Range: 0-65
Dryness: 25 or below
Optimum: 35-45
High level of hydration: 55+





Bio-Therapeutic

Skin Care for life inside and out®



SA-TM-600 REV061016

WARNINGS

- Small parts may pose as a choking hazard to children. Keep the bt-analyze away from children at all times.
- Do not use the bt-analyze on broken skin.
- Do not immerse in liquid.
- Follow proper cleaning methods as outlined in the maintenance portion of this manual.
- Inspect prior to each use, and discontinue use if the membrane is damaged.
- Do not open the unit for any reason; doing so will void the warranty.
- Keep the membrane covered while not in use.
- Discontinue use if bt-analyze appears damaged in any way. The bt-analyze® contains no serviceable parts. Refer to 'Guarantee and Service' if your bt-analyze no longer works or needs repair.
- Do not use attachments other than those recommended by the manufacturer.
- Keep away from heated surfaces.
- This appliance is not intended for use by children. Children should be supervised to ensure that they do not play with the bt-analyze.
- The bt-analyze is designed for cosmetic use only.
- Use the bt-analyze only for its intended use as described in this booklet.

LIABILITY TERMS

Bio-Therapeutic Inc./Microcurrent Technology Inc. (BTI/MCTI) shall be responsible for the proper mechanical operation of the bt-analyze during the warranty period under the following terms: Technical repairs must be preformed by personnel expressly authorized by BTI/MCTI only.

The bt-analyze must be used only in the manner herein described.

Operators shall maintain sole responsibility for proper sanitization and timely replacement of bt-analyze disposable products and parts. BTI/MCTI has no control over the methods by which operators service clients. Therefore, BTI/MCTI's sole responsibility shall be limited to repair and/or replacement of unit ONLY under specification set forth above. BTI/MCTI assumes no liability with regards to application. All application protocol recommendations described in this manual are provided for reference purposes only. Professionals must analyze and research individual client characteristics prior to recommending service to determine appropriate protocol. Buyer/operator herein agrees to this limited liability as a condition of sale.

MAINTENANCE

The bt-analyze should be kept free of liquid, product build up, and debris at all times. The bt-analyze should be cleaned with a damp cloth and mild cleanser. Only the membrane should be sanitized with alcohol. Harsh disinfectants and alcohol can damage the case of the bt-analyze and should never be used on the plastic components of the unit. The unit should never be immersed in liquid. Signs of liquid, product, corrosion, etc inside the unit will void the warranty.

The bt-analyze should be stored in a safe area at all times. The unit is built to the highest quality standards, however treat it with care. Avoid dropping the unit or damaging the screen or membrane. Discontinue use if damage occurs. The bt-analyze requires 2 AAA batteries. Batteries should be replaced as needed by sliding the battery cover downward, located opposite the LCD display.

WARRANTY

Bio-Therapeutic guarantees the bt-analyze for 1 year after the date of purchase. Defects due to faulty materials and workmanship will be repaired or replaced by Bio-Therapeutic in the qualifying period is provided. Bio-Therapeutic does not guarantee the bt-analyze of water/liquid damage, normal wear and tear including chips, scratches, abrasions, discoloration, fading, or damage determined by Bio-Therapeutic as misuse, abuse, neglect, alterations, or unauthorized repair.

CONTRAINDICATIONS

Pacemaker or other implanted electrical device
Cancer
Pregnancy
Any contagious or transmittable diseases
Consult a Physician before using if there are any health or medical concerns

TECHNICAL FEATURES

- Dimensions: 3 1/4" x 3 1/4" x 3/4"
- Weight: 1.5 oz. (no batteries)
- Power Supply: 2 AAA batteries